This article is intended for exclusive use by healthcare professionals. The device must be used in accordance with the procedures of Good Clinical Practice, in conformity with the protocols and guidelines.

Detailed composition is present in labelling

Special operating instructions are provided for the component(s):

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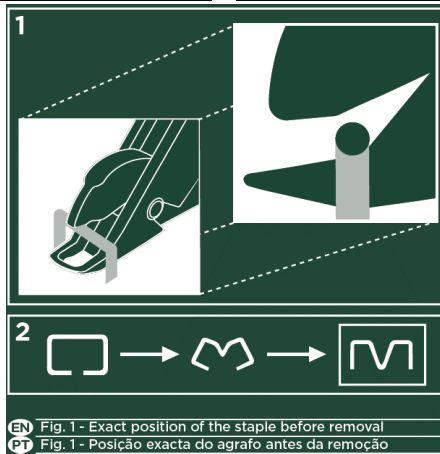
Removedor de agrafos

INTENDED PURPOSE

Skin staple remover is intended for removal of surgical skin staples.

PRODUCT INFORMATION

- Fast and atraumatic removal of surgical skin staples.
- Functional for the intended use made of light high performance polymer.
- Adequate mechanical properties and resistance.
- Does not require maintenance, repair or reprocessing in hospital management.
- Environmentally friendly as savings on water and detergent consumption influence carbon footprint. Simple and safe incineration through hospital waste disposal systems.

INSTRUCTIONS FOR USE

- 1) Inspect the staples and make sure they are perpendicular to skin before removal.
- 2) Place the lower jaw of staple remover between staple and skin (figure 1).
- 3) Squeeze and lift the staple remover perpendicularly to the skin.
- 4) The staple will be bent open by squeezing (figure 2).
- 5) Lift the open staple straight up from the tissue and discard according to local requirements.

WARNINGS AND PRECAUTIONS

- Skin staple remover is to be used exclusively by healthcare professionals.
- Carefully respect the positioning of the skin staple remover according with the instructions for use, in order to avoid the risk of pain or skin damage.
- Do not re-use. Skin staple remover is intended to be used on a single patient during one single procedure to prevent risks from contamination, including possible infections, and/or functional damage.
- Visually inspect the sterile barrier system and the device prior to use. Do not use the device if you detect any breach in packaging integrity or any sign of the device deterioration that may compromise its use and/or functionality.
- Dispose according to local legal regulations for healthcare waste.

The occurrence of serious incidents is not expected with the use of skin staple remover, but we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

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CONTRAINDICATIONS AND/ OR PRECAUTIONS

Store in a dry place at room temperature and protected from sunlight.

Do not use the contents if the package is opened or damaged. Do not re-sterilize. Do not reuse.

There are no other known contraindications and/or warnings, in addition to the above, specific for the component(s) in question.

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