Umbilical cord clamp

Clamp para cordão umbilical

- FR Clamp de Bar ombilical
- DE Nabelschnurklemme
- NL Navelstrengklem
- IT Morsetto per cordone ombelicale
- ES Pinza para cordón umbilical
- SV Navelsträngsklamma
- GR Κλάμμα za πύλινα βρύση
- HR Stezaljka za pupčanu vrpču
- CZ Svoraka na pupčnický šňůru
- DA Navelstrengsklemmer
- FI Nabanõõri klamber
- ET Napanõõri puristin
- EL Λαβίδα ομφάλιου λώρου
- HU Kétkézkeszínócsipesz
- LV Nabassaites spailes
- LT Virkėtelės spaustukės
- PL Zaczisk do powpinowy
- NO Clemis cordon ombilical
- SK Svoraka na pupčnický šnúru
- SI Spunka za popkovino
- NO Navelstrengsklemme



IMPORTANT

CAUTION: Incorrect placement of the clamp on the umbilical cord may lead to an inadequate seal and consequent blood loss that might prove fatal for some babies. **Read the instructions before using the device.**

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https://www.youtube.com/watch?v=XXVUPjXaiBQ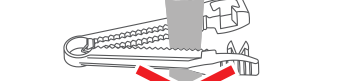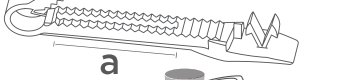

EN UMBILICAL CORD CLAMP
INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT
Umbilical cord clamp is intended to clamp the umbilical cord after delivery to prevent bleeding from the cord stump after the cord has been cut.
INSTRUCTIONS FOR USE
1-Check the thickness of the umbilical cord. In case it is a thin cord, it may be necessary to fold it before placing the clamp to obtain a bigger resistance to the clamp closing, or consider applying two clamps.
2-Place the cord within the clamp, in the clamping zone (figure 1a).
3-Check that there is not any object blocking the locking site (e.g. swabs, surgical instruments, etc.).
4-Close the clamp until it locks and clicks (figure 2). Make sure that the clamp is well closed, with both teeth correctly in place (figure 2) and that it is impossible to open it with the hands. Cut the umbilical cord.
5-Confirm that there is no bleeding at the end of the cord. Check regularly for signs of bleeding due to a misplaced clamp or due to a damaged cord.
WARNINGS AND PRECAUTIONS
Intended to be used exclusively by paediatric/neonatology healthcare professionals.
Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
Visually inspect the device. Do not use the device if you detect any deformation or defect that may compromise the performance.
Always check the integrity of the locking system before use. Do not use the device if the locking system is damaged.
Placement of the clamp on the umbilical cord may lead to an inadequate seal and consequent blood loss that might prove fatal.
Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infection.
When used on healthcare facilities, and if contaminated: Dispose according to applicable legislation and/or infection prevention standards to avoid contamination.
Other risks: Dispose according to applicable legislation and/or infection prevention standards for human waste.
The occurrence of serious incidents is not expected with the use of the device. In case of a serious incident, please contact the manufacturer (UE) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent damage, should be notified to Bastos Viegas and to the Competent Authority of your country.

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