EN Compression bandages

416 - Medium compression bandages STANDARD
417 - Strong compression bandages STRONG

INTENDED PURPOSE


Compression bandages are intended for compression therapy, to reduce oedema and treatment of venous disorders.

INSTRUCTIONS FOR USE

- It is recommended to apply the bandage when swelling is down, preferably in the morning or after some rest in a raised position.
- Unless it is contraindicated, ensure that the foot is at a 90° angle and start bandaging according with adequate bandaging technique (examples of bandaging technique on fig.1).
- Apply the bandage holding the roll close to the limb and with the rolled-up part facing outward.
- Apply the bandage so that the pressure decreases from the foot to the knee.
- Use a secondary dressing (e.g. adhesive) or a clip fastener to secure the bandage in place.

WASHING INSTRUCTIONS

Compression bandages may be re-washed until a maximum of 3 times, according to the defined instructions, in order to maintain their elasticity properties.

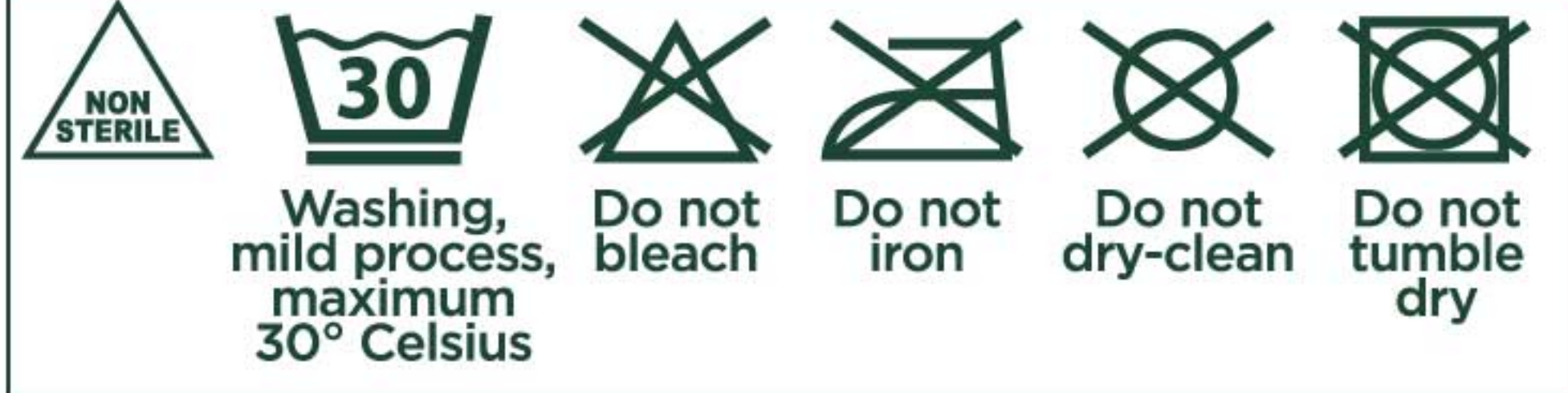WARNINGS AND PRECAUTIONS

- Compression bandages are to be used exclusively by healthcare professionals. An excessive pressure may cause damage to the limbs (e.g. skin necrosis or nerve injuries) and a deficient pressure compromises the treatment.
- Do not cut the bandage.
- After application, pain should progressively decrease. If pain increases or a new one occurs, which do not disappear while walking around, the bandage must be removed immediately.
- Do not use the device if you detect any sign of deterioration that may compromise its use and/or functionality.
- When used on healthcare facilities, and if contaminated, dispose according to local legal regulations for healthcare waste. Otherwise, dispose according to local legal regulations for urban waste.

The occurrence of serious incidents is not expected with the use of compression bandages if applied correctly, but we must inform that serious incidents related with the use of the device, that are life-threatening or compromises health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

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