Compression bandages

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FR Bandes compressives

DE Dauerelastische Binden

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SV Kompressionsbinda

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www.bastosviegas.com

EN COMPRESSION BANDAGES

416 Medium compression bandages⁴⁾
417 Strong compression bandages⁵⁾

INTENDED PURPOSE AND INDICATIONS

Compression bandages are intended for compression therapy, to reduce oedema and treatment of venous disorders. May also be used for retention of dressings.

CLINICAL BENEFIT

-To reduce oedema and fluid accumulation.
-To promote wound healing and to reduce risk of infection by retention of dressings.

INSTRUCTIONS FOR USE

1-It is recommended to apply the bandage when swelling is down, preferably in the morning or after some rest in a raised position.
2-Unless it is contraindicated, ensure that the foot is at a 90° angle and start bandaging according with adequate bandaging technique (examples of bandaging technique on fig.).
3-Apply the bandage holding the roll close to the limb and with the rolled-up part facing outward.
4-Apply the bandage so that the pressure decreases from the foot to the knee.
5-Use a secondary dressing (e.g. adhesive) or a clip fastener to secure the bandage in place.

WASHING INSTRUCTIONS

Compression bandages may be re-washed until a maximum of 3 times, according to the instructions below, in order to maintain their elasticity properties.

WARNINGS AND PRECAUTIONS

-Intended to be used exclusively by healthcare professionals.
-Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
-Do not cut the bandage.
-An excessive pressure may cause damage to the limbs (e.g. skin necrosis or nerve injuries) and a deficient pressure compromises the treatment.
-After application, pain should progressively decrease. If pain increases or a new one occurs, which do not disappear while walking around, the bandage must be removed immediately.
-When used on healthcare facilities, and if contaminated: Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.
Otherwise: Dispose according to applicable legislation and/or infection prevention standards for urban waste.

The occurrence of serious incidents is not expected with the use of the device if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

LABELLING SYMBOLS

1)UKCA marking 2)United Kingdom authorized representative

3)Paper (identification of packaging material)

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