Surgical containers

Recipientes cirúrgicos

FR Conteneurs à usage chirurgical

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NL Chirurgische containers

IT Contenitori chirurgici

ES Contenedores quirúrgico

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EN Surgical containers

DEVICES

-Surgical bowls; basins and rectangular basins);
-Trays (includes kidney trays, basins liners;
-catheterization trays,
laparoscopic/endoscopic trays and trays with guide wire;
-Basins (includes ring basins, placenta

DEVICE INFORMATION

Wide-range of single-use surgical containers comply with the highest quality standards. Medical devices classified as Class Ila devices due to their broad approved indications for use, including use during invasive procedures. Some references present an appropriate lid as well as adhesives for fixation to surfaces. Healthcare professionals should select a device with a suitable capacity and design appropriate for the intended purpose.

INTENDED PURPOSE AND INDICATIONS

Surgical bowls, trays, basins, basin liners and pitchers:
-To temporarily hold organs or tissues.
-To hold or for preparation of medical devices such as catheters with medicinal solutions.
-To hold saline solution or saline solution with contrast media or heparin intended for injection.
-To hold aqueous, saline solutions for irrigation.
-To hold aqueous, saline or antiseptic solutions for wound cleansing.
-To hold antiseptics for disinfection of skin or mucosa.
Additionally, and due to their design, are intended to be used:
Pitchers - to pour solutions.
Kidney trays, catheterization trays (boat shape) - to fit the human body, are adequate to collect body fluids.
Trays with guidewire bowls - to prepare and to temporary store a guidewire while keeping it in place and hydrated.
Basin liners - The basin liner, designed with an integrated drape, is used to prevent the infection of the patient in a surgical context. Shaped to fit the interior of a ring basin, the liner includes a drape that covers the legs of the ring stand. Additionally, it serves to hold warm irrigation solutions.
Surgical bowl lids and Basin lids:
-To obtain the closed version of surgical bowls or basins during transport, storage or disposal.
Placenta basins:
-Specifically to temporarily hold placenta and/or umbilical cord for intact preservation and for subsequent preparation of placental membrane which can after be used in a context such as woundcare.

WARNINGS AND PRECAUTIONS

-Not intended to be used for preparation of bone cement.
-Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
-Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
-Before use, check visually for particulate matter and use only if the contents are clear.
-For the preparation of injectables use of a filter needle. Be aware that open containers may increase the risk of contamination.
-For ophthalmic invasive procedures, flush the surgical container with sterile solution before use.
-Intended to be used exclusively by healthcare professionals.
-Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.
-Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.

The occurrence of serious incidents is not expected with the use of the device if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromising health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

LABELLING SYMBOLS

1)Representative authorized representative 2)Paper (identification of packaging material)

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