Basin liner

Cobertura de bacia

REF 4886-021

REF 4886-022

FR Pochette de bassin
DE Abdecktuch für Waschbecken
NL Afdekking voor spoelkom
IT Telo copri lavandino
ES Revestimiento para recipientes
SV Skydd till handfat
BG Покривало за съд за опасни отпадъци
HR Prekrivka za posudu za otopine
CS Rouška na překrytí nádoby
DA Overtræk med plastikstykke beregnet til skål
ET Valamukatte
FI Instrumenttiastian suojaaliina
EL Επένδυση λεκάνης
HU Mosdatótál-takaró
LV Mediciniskās mazgātnes pārklājs
LT Medicinių atliekų dubens apklotas
PL Wyściółka pojemnika
RO Acoperitoare bazin
SK Výstelka pre miskú
SL Pregrinjalo za medicinsko opremo
NO Servantforing

UKRP ¹⁾

ZILION ⁴⁾

STERILE EO

CE 1639

MD

Bastos Viegas,s.a.
Avenida da Fábrica, 298. 4560-164 Guilhufe, Penafiel- Portugal
Tel.: 255 729 500 - geral@bastosviegas.com
www.bastosviegas.com

IFU551999124. Rev. 0 (25-09-2025)

IT TELO COPRI LAVANDINO

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EN BASIN LINER

INTENDED PURPOSE AND INDICATIONS

-To temporarily hold organs or tissues.
-To hold or for preparation of medical devices such as catheters with medicinal solutions.
-To hold aqueous, saline solutions for irrigation.

Additionally, and due to their design, are intended to be used:
The basin liner, designed with an integrated drape, is used to prevent the infectious agents transfer in a surgical context.
Shaped to fit the interior of a ring basin, the liner includes a drape that covers the legs of the ring stand. Additionally, it serves to hold warm irrigation solutions.

WARNINGS AND PRECAUTIONS

-Not intended to be used for preparation of injectables nor bone cement.
-Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
-Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
-Before use, check visually for particulate matter and use only if the contents are clear.
-For ophthalmic invasive procedures, flush the surgical container with sterile solution before use.
-Intended to be used exclusively by healthcare professionals.
-Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.
-Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.

The occurrence of serious incidents is not expected with the use of the device if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

LABELLING SYMBOLS

1)United Kingdom authorized representative 2)Paper (identification of packaging material)
3)Opening direction 4)Trademark

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